**EDS** ESSENTIAL DENTAL SYSTEMS

CE 0086

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.

# EZ-Fit™

Multi-Tiered Passive Fiber Post

- Size 0 Posts
- Size 1 Posts
- Size 2 Posts

Essential Dental Systems, Inc.  
89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

## Sterilization Procedure:

Posts, Drills and Wrenches are non-sterile. Prior to use, bag materials and sterilize with a fractionated vacuum autoclave or gravity displacement autoclave for 20 minutes at 121°C. When reprocessing drills, wrenches, or taps, remove debris and dry before sterilization. Remove debris with an ultrasonic cleaning unit for 5 minutes or a thermal disinfectant unit (washer-disinfectant) at 80°C for 10 minutes. For complete instructions see website.

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**ENGLISH:** Post selection is best done by placing a post in a hemostat and holding it over an undistorted x-ray. If there is a minimum of approximately 0.5mm of lateral root structure at the most apical placement of the post, there is sufficient root structure for placement. Make sure to use a large enough post size (i.e. diameter) to withstand the occlusal forces. Note that the EZ-Fit Post is designed with a second tier for reduction of apical stress. The second tier provides increased retention at the critical coronal level (where the canal flares), stabilizing the post for a long lasting restoration.

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**1. ENGLISH:** Use a Peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal (EDS' Gates Glidden Drills are recommended). Then use the primary reamer.

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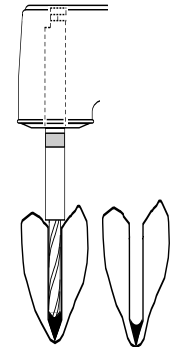
**2. ENGLISH:** Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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
**3. ENGLISH:** Use the secondary drill to prepare the seat for the second tier. (The second tier must be fully seated. If not, you reduce the post's retention and increase the chance of fracture under function.)

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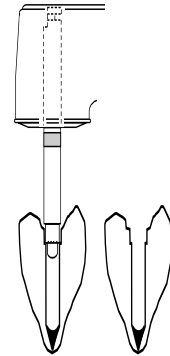
**4. ENGLISH:** Trial insert the post to determine fit. Note second tier must fully seat.

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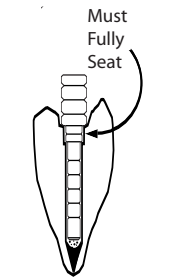
Over

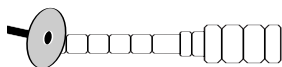**5. ENGLISH:** Post adjustment: cut apical end, if necessary, to ensure seating of the second tier. Note: apical adjustment can be done after trial insertion.

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**6. ENGLISH:** Note: For optimal retention use a luting cement with compatible bonding agent.

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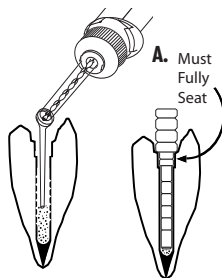
**7. ENGLISH:** Dry the canal. Place composite resin cement in the canal. **Note fully seated second tier (A).** (Flexi-Flow Cem<sup>®</sup> from EDS is recommended). Follow with core formation and preparation (Ti-Core<sup>®</sup> composite Core build-up material from EDS is recommended).

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Notes: Keep posts out of sunlight. Prolonged direct exposure to sunlight will cause the post to turn brown. In all EZ-Fit post restorations, a minimum of a 1.5mm ferrule around the tooth is needed for the final restoration.

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**FOR ADDITIONAL PRODUCT INFORMATION  
AND LATEST INSTRUCTIONS, PLEASE VISIT  
WWW.EDSDENTAL.COM  
OR TO ORDER  
CALL YOUR DENTAL DEALER**

EC REP

Authorized European Representative:

Regulatory Affairs ONLY:

European Device Solutions  
33, Stapleford Road,  
Middlesbrough, Cleveland,  
United Kingdom.  
Contact E-Mail: TQMUK@aol.com

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

**1-800-22-FLEXI**  
Tel: (201) 487-9090  
Fax: (201) 487-5120  
www.edsdental.com

EZFITN-1

MADE IN USA  
U.S. Patent No. 5,669,772 and U.S. and other foreign patents pending.