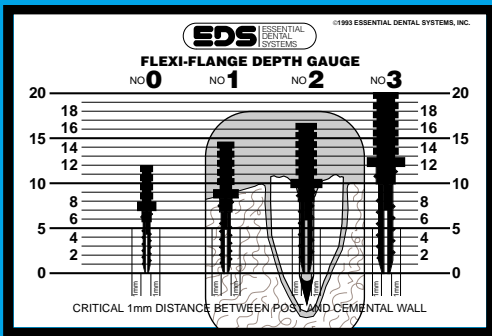
**NOTE:** If you are using Flexi-Flange® for the first time, you need the wrench, depth gauge, and extender which are included in the Introductory Kit (Cat. No. 410-00 or 415-00), or may be purchased separately.

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## 1.

**ENGLISH:** Use a depth gauge to determine optimum depth of the post. Flange of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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## 2.

**ENGLISH:** Use a peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal. (EDS' Gates Glidden Drills are recommended.) Then use primary reamer.

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## 3.

**ENGLISH:** Use the primary reamer to prepare the full length of the post hole. (Cut Wet).

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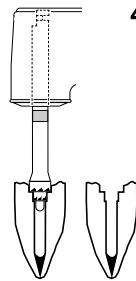
9800  
CE

ESSENTIAL DENTAL SYSTEMS

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.


**4. ENGLISH:** Use a countersink drill/root facer to create the flange and second tier preparations. (The flange must be fully seated. If not, you reduce the post's stability and retention, and increase the chance of fracture under function.)

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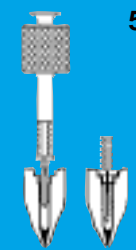
**5. ENGLISH:** Trial insert the post to determine fit.

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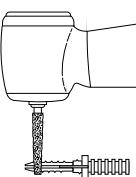**6. ENGLISH:** Post adjustment: cut apical end, if necessary, to ensure seating of the flange. (Cut both legs at the same time.) Note: apical adjustment is always done after trial insertion.

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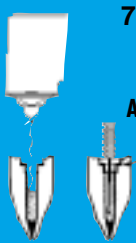
**7. ENGLISH:** Post cementation. **Note fully seated flange, (A).** (Flexi-Flow® Cat. No. 850-00 or Flexi-Flow® Natural Cat. No. 860-00 cement from EDS is recommended.) Follow with core formation and preparation (Ti-Core® Cat. No. 800-00 or Ti-Core® Natural Cat. No. 810-00 composite Core build-up material from EDS is recommended).

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FOR MORE INFORMATION ON THESE AND ALL OUR INNOVATIVE PRODUCTS, OR TO ORDER CALL YOUR LOCAL DENTAL DEALER

**EDS** ESSENTIAL DENTAL SYSTEMS

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

**1-800-22-FLEXI**

Tel: (201) 487-9090

Fax: (201) 487-5120

MADE IN USA

U.S. Patent No. Re.31,948 and U.S. and other Foreign Patents Pending.