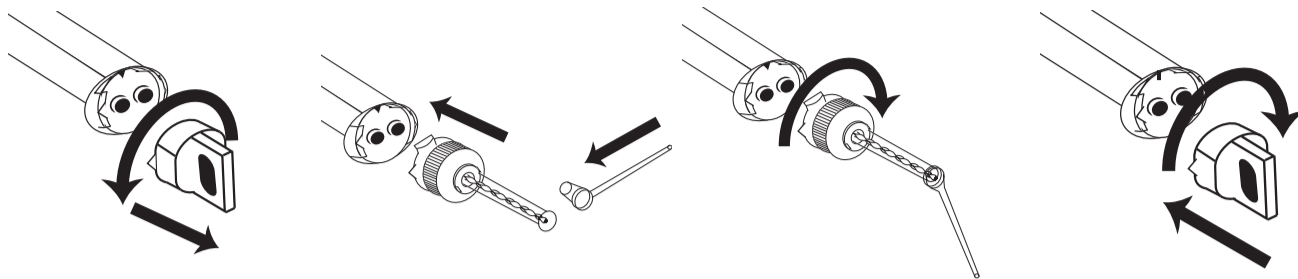
Auto™ Flexi-Flow

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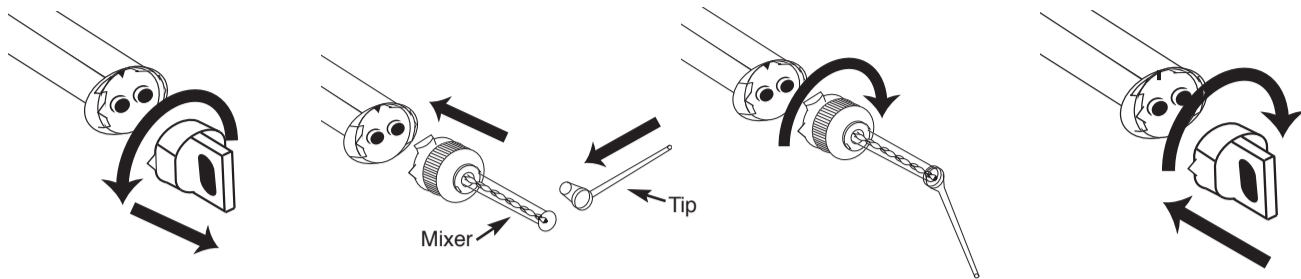
Flexi-Flow Auto™ E

Titanium and Lanthanide Self Curing, Reinforced Composite Resin Cements
Recommended for all prefabricated post systems.

Clinical Technique:

1. Prepare the post-hole in order to receive the cement.
2. Remove syringe cap, place the mixing stator on the syringe and place the post-hole delivery tip onto the mixing stator (Figures 1 - 4).

1. Before attaching the mixing tip, remove the syringe cap by turning 90° counter clockwise and pulling to remove.
 2. Push the mixing tip onto the syringe. Line up the notch on the tip with the corresponding notch on the syringe.
 3. Turn the tip 90° clockwise until it locks into position.
 4. Remove and discard the mixing tip after use and replace the syringe cap.
3. As pressure is applied to the plunger the base and catalyst are extruded through the stator and mixed automatically. This guarantees a consistent mix each and every time.

4. Place the post-hole delivery tip into the post-hole and express Flexi-Flow Auto and/or Flexi-Flow Auto E cements directly into the post-hole (Figure #5).
5. In order to achieve maximum penetration depth of the cement into the preparation, use a lentulo spiral to spin the cement to the apical end of the post-hole (Figure #6).
6. The cement sets according to the following schedule:

Self Curing Procedural Timing

Working Time (Gel Time): 3:00 min

Setting Time: 5:00 min

7. Coat and insert post, let the cement set undisturbed for the next 4 minutes. The post is now ready for core build-up. Note: To make use of the maximum tensile strength of Flexi-Flow Auto and Flexi-Flow Auto E, the post-hole should be etched or grooved (use Essential Dental Systems' Grooving Drills, Cat. No. 593-0 or Cat. No. 593-123) when using a passive post.

Note: Flexi-Flow should **not** be used with any self curing bonding agents. Self curing bonding agents greatly accelerate the setting time of Flexi-Flow Auto and Flexi-Flow Auto E.

Storage:

Store between 60°F (15°C) - 85°F (29°C)

MSDS sheets available. Call: 1800-22-FLEXI or 201-487-9090. E-mail: info@edsdental.com.

Precautions:

- Contact with eyes and skin should be avoided. If accidental contact occurs rinse immediately with water and consult a physician if necessary.
- Do not use materials containing eugenol it will delay or inhibit the set.
- Water and oily air inhibit polymerization.
- A dry field is indicated when placing a composite.
- There are no known systemic side effects to date, however, do not use in patients with known sensitivities to Bis-GMA, benzyl peroxide or composite resin materials.

Flexi-Flow Auto™ E

Self Curing Reinforced Cement

Tooth Color Vita® A2

REF
Cat. No. 880-00

EC REP

Authorized European Representative:

Regulatory Affairs, O.H., Y.

Essential Dental Solutions

33, Stapleford Road,

Middleborough, Dorsetland,

United Kingdom

Contact E-Mail: TGMUK@aad.com

STORE BETWEEN

60°-85°F

15°-29°C

201
15°C

85°F
30°C

Caution: do not
insert into the
mouth

Do not use

CE

0086

EDS ESSENTIAL
DENTAL
SYSTEMS

89 Leaning Street
S. Hackensack, NJ 07606

Phone: 1-800-22-FLEXI • 201-487-9090

Fax: 201-487-5120

www.edsdental.com

Caution: Federal law restricts this device to sale by or on the order of a licensed dentist.
U.S. Patent Nos. 5,034,433 & 5,204,398.
European Patent No. EP 0 631 598 B1. Other patents pending.

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