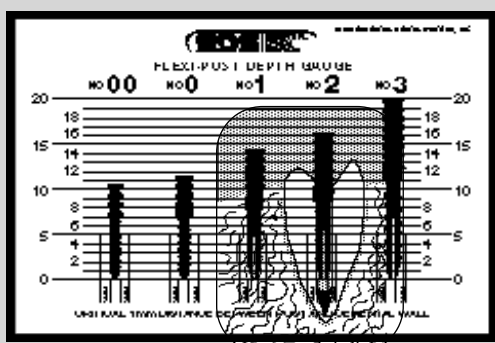
NOTE: If you are using Flexi-Post® for the first time, you need the wrenches, depth gauge and extender which are included in the Introductory Kit (Cat. No. 110-00 or 110-01), or may be purchased separately.

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1.

ENGLISH: Use a depth gauge to determine optimum depth of the post. The second tier of the post must be fully seated to achieve maximum stability, retention and resistance to fracture.

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2.

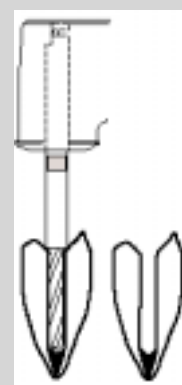
ENGLISH: Use a Peeso or Essential Gates Glidden drill to remove gutta percha, and for preliminary sizing of the canal (EDS' Gates Glidden Drills are recommended). Then use the primary reamer.

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3.

ENGLISH: Use the primary reamer to prepare the full length of the post hole (Cut Wet).

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Caution: Federal law
restricts the sale of
this product to the order of
a licensed dentist.

EDS ESSENTIAL DENTAL SYSTEMS

4. ENGLISH: Use the secondary drill to prepare the seat for the second tier. (The second tier must be fully seated. If not, you reduce the Flexi-Post's retention and increase the chance of fracture under function.)

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5. ENGLISH: Trial insert the post to determine fit.

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6. ENGLISH: Post adjustment: cut apical end, if necessary, to ensure seating of the second tier. (Cut both legs at the same time). Note: apical adjustment is always done after trial insertion.

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7. ENGLISH: Post cementation. **Note fully seated second tier (A).** (Flexi-Flow® Cat. No. 850-00 or Flexi-Flow® Natural Cat. No. 860-00 cement from EDS is recommended). Follow with core formation and preparation (Ti-Core® Cat. No. 800-00 or Ti-Core® Natural Cat. No. 810-00 composite Core build-up material from EDS is recommended).

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FOR MORE INFORMATION ON THESE AND ALL OUR INNOVATIVE PRODUCTS, OR TO ORDER CALL YOUR LOCAL DENTAL DEALER

EDS ESSENTIAL DENTAL SYSTEMS

89 Leuning Street, S. Hackensack, NJ 07606, U.S.A.

1-800-22-FLEXI

Tel: (201) 487-9090

Fax: (201) 487-5120

MADE IN USA
U.S. Patent No. Re.31,948 and U.S. and other foreign patents pending.