# TI-CORE®

## Contrasting Gray

### Core Material

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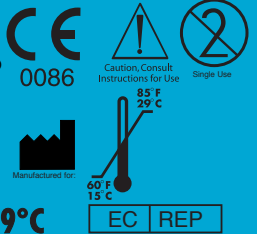
# 805-00

### CONTENTS:

- ONE 15 GRAM JAR OF BASE
- ONE 15 GRAM JAR OF CATALYST
- 50 DISPOSABLE SPATULAS
- 100 MIXING PAD SHEETS
- INSTRUCTION BOOKLET

2012 © Essential Dental Systems, Inc.  
89 Leuning Street, S. Hackensack, NJ 07606  
www.edsdental.com

**STORE BETWEEN 60°-85°F / 15°-29°C**

Caution: Federal law restricts this device to sale by  
or on the order of a licensed dentist.  
U.S. Patent Nos. 5,034,433 & 5,204,398.  
European Patent No. EP 0 631 598 B1. Other patents pending.

Authorized European Representative:  
Regulatory Affairs ONLY,  
European Device Solutions  
33, Stapleford Road,  
Middlesbrough, Cleveland,  
United Kingdom.  
Contact E-Mail: TOMUK@aol.com

### English:

Ti-Core is a reinforced, autocuring composite, that creates a unique combination of great strength and aesthetics. The addition of patented Ti-Core means the elimination of ditching during core preparation, and long term success of the restoration under function.

### Precautions:

- Contact with eyes and skin should be avoided. If accidental contact occurs rinse immediately with water and consult a physician if necessary.
- Do not use materials containing eugenol it will delay or inhibit the set.
- Water and oily air inhibit polymerization.
- There are no known systemic side effects to date, however, do not use in patients with known sensitivities to Bis-GMA, benzyl peroxide or composite resin materials.

### AREAS OF USE:

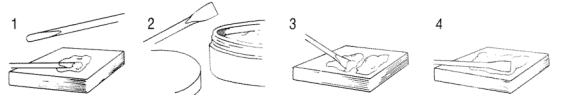
1. NONVITAL TEETH - Ti-Core is recommended for use with all prefabricated posts and/or pin build-ups.
2. VITAL TEETH - Ti-Core may be used with pins to build-up missing tooth structure. A calcium hydroxide or glass ionomer base may be used under the Ti-Core Natural composite.

### BONDING TO ENAMEL & DENTIN:

- a. Ti-Core can be bonded to enamel with the acid etch technique.
- b. Ti-Core Natural can be bonded to dentin using any of the available multi-step dentin bonding agents currently on the market.

### MIXING TI-CORE:

1. Using the rounded end of the disposable plastic spatula, remove from the jar of CATALYST PASTE an amount slightly less than the volume of the cavity to be filled. Place on mixing pad.
2. Using the flat, squared end of the disposable plastic spatula, remove from the jar of BASE PASTE an amount approximately equal to the Catalyst Paste placed on the mixing pad in step 1.
3. Place Base Paste next to Catalyst Paste on mixing pad.  
REPLACE CAPS ON JARS.
4. Using the flat, squared end of the disposable plastic spatula, mix the two pastes for about 20 seconds using 1 to 2 strokes per second. The objective is to make the mixture as homogenous as possible.
5. **To make Ti-Core set faster, add more base. To make Ti-Core set slower, add more catalyst.**

### PLACEMENT:

1. To create an intimate adaptation between root, post and core material, place Ti-Core into a core form and seat over the root and post with firm pressure.
2. Alternatively, you can place a matrix band around the prepared tooth and post, syringe Ti-Core and apply light pressure for an intimate fit.

### TOOTH PREPARATION:

After three minutes from the beginning of mixing the core build-up material, remove the core form or matrix band. Diamond or carbide burs may be used to prepare the tooth. Air and water cooling is recommended to cut the material more efficiently.

MIX TIME	PLACEMENT TIME	SETTING TIME (DO NOT DISTURB)	HARDENING TIME	REMOVE MATRIX, FINISH AND POLISH
20 SEC.	60 SEC.	40 SEC.	1 MIN.	
0		1.20	2	3

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